[SE] Bruksanvisning

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[EN] Instructions

Purpose and protection

The product is intended to protect against drip contamination.

Storage before use

Store the product in its carton. Avoid exposing the product to direct sunlight during storage. If the protective film is removed, The product is then sensitive to scratches.

Use

- Remove the protective film on both sides of the visor before use.
- Open the velcro straps.
- Place the visor against the forehead and fasten the visor around the head by closing the velcro straps
- Removed in reverse order.

Storage between uses

The product is sensitive to scratches after the protective film has been removed.

Damage, maintenance and service

Damaged product should be discarded. Otherwise, the product requires no maintenance.

Disinfection and cleaning

The product is not sterile. Clean with:

- Water: Lukewarm water with the addition of a neutral detergent. Rinse afterwards with clean water and dry with a soft sponge, wool cloth or similar.
 - Disinfection: Surface disinfection (Ethanol 50–80%, isopropanol 5–10%)
- Never use sharp objects to close or dirt.

Reuse

When the product loses its transparency or not feels hygienic, it should be discarded.

Certification

Tested according to MET5659 Issue 1, PPER (EU) 2016/425

Cassation period

This product is valid to use until 31 dec 2020.

Recycling

Sorted as plastic packaging. Contaminated product must be sorted according to the employer's instructions.

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
SE556705385401
info@packing.se
+46 (0) 227-312 55

Certification

The EU type control is carried out by RISE Research Institutes of Sweden AB
Box 857, SE-50115 BORÅS
Notified No. 0402

Marking

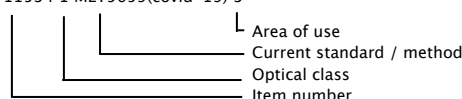ART11534 1 MET5659(covid-19) 3

This manual should be made free of charge for each person.

The EU declaration of conformity can be downloaded at this address:
https://www.packing.se/skyddsvisir/eu_forsakran.pdf

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Packing Kungsör AB